

Ipas 3MM Cannula

A Cost-Effective Solution for
Endometrial Biopsy



Product Code: EB3

PRODUCT DESCRIPTION



Single use and sterile packed



Semi-rigid flexibility



Permanently integrated winged
base



Color-coded by size



Depth markings printed in cm



3mm- two "whistle cut" openings



Approximately 23 cm (9 in)
in length



Latex-free plastic

INSTRUCTION FOR USE

- Prepare patient and offer pain relief – oral analgesia, paracervical block.
- Creating the vacuum on the aspirator by pressing valve buttons and pulling plunger.
- Insert cannula into the uterus and attach prepared aspirator.
- Release valve buttons on the aspirator and gently rotate to sample tissue.

COMPATIBILITY WITH ASPIRATORS

- Compatible with Ipas MVA Plus®, Ipas Double-Valve Aspirator with a 6 mm adapter
- Compatible with Ipas Single-Valve Aspirator without an adapter

INDICATIONS FOR USE

- Evaluation of abnormal uterine bleeding - heavy menstrual bleeding, intermenstrual bleeding, and post coital bleeding
- Evaluation of postmenopausal bleeding

BENEFITS

Convenience

- Portable, quiet and easily stored

Safe

- Fully sterilized