Ipαs Flexible Karman Cannulae



A cost-effective solution for uterine evacuation. The original flexible cannulae.

PRODUCT DESCRIPTION



Single use device



Ready to use and sterilely packed



Printing on cannulae color-coded by size



Uterine depth markers from 6 to 11 cm printed on cannulae

INSTRUCTION FOR USE

- Providers select the cannula size based on the woman's gestational age.
- Prepare patient and provide pain relief oral analgesia, paracervical block.
- Creating the vacuum on the aspirator by pressing valve buttons and pulling plunger.
- Insert cannula into the uterus and attach prepared aspirator.
- Release valve buttons on the aspirator and gently rotate to aspirate uterine contents.

INDICATIONS FOR USE

- Miscarriage management
- Induced or incomplete abortion up to 12 weeks since the woman's last menstrual period
- Endometrial biopsy

Product Code: FC4 to FC10, and FC12



Sized 4 mm to 8 mm - two "whistle cut" openings



Sized 9 mm, 10 mm, and 12 mm - one large "scoop cut" opening



Approximately 24 cm (9.5 in) in length



Latex-free plastic

COMPATIBILITY WITH ASPIRATORS

- Size 4-10 mm compatible with Ipas MVA Plus[®] and Ipas Double-Valve Aspirator with adapter
- Size 12mm compatible with Ipas MVA Plus[®] and Ipas Double-Valve Aspirator
- Size 4-6mm compatible with Ipas Single-Valve Aspirator

BENEFITS

Flexible

• More flexible cannulae which is the best option for retroverted or retroflexed uterine position

Practical

• Feel the 'gritty' sensation that indicates an evacuated uterus

Easy to identify

- COLOR CODE
- Cannulae are color-coded and use adapters with the same color-coding

4mm

PRODUCT CODE/DIAMETER CHART

 Size
 4 mm
 5 mm
 6 mm
 7 mm
 8 mm
 9 mm
 10 mm
 12 mm

 Code
 FC4
 FC5
 FC6
 FC7
 FC8
 FC9
 FC10
 FC12

WomanCare Global Email: contact@dktwomancare.org www.dktwomancare.org

