Ipαs EasyGrip[®] Cannulae



Ipas EasyGrip[®] Cannulae is an effective solution for uterine evacuation.The tactile response of a rigid curette with the gentle probe of a flexible cannulae.

PRODUCT DESCRIPTION



Single-use device



Ready to use and sterilely packed

Permanently-integrated winged base



Semi-rigid flexibility



Color-coded by size

INSTRUCTION FOR USE

- Providers select the cannula size based on the woman's gestational age.
- Prepare patient and provide pain relief oral analgesia, paracervical block.
- Creating the vacuum on the aspirator by pressing valve buttons and pulling plunger.
- Insert cannula into the uterus and attach prepared aspirator.
- Release valve buttons on the aspirator and gently rotate to aspirate uterine contents.

INDICATIONS FOR USE

- Miscarriage management
- Induced or incomplete abortion up to 12 weeks since the woman's last menstrual period
- Endometrial biopsy

Product Code: SR4-SU to SR10-SU, and SR12-SU



Uterine depth markers from 6 to 11 cm printed on cannulae



Sized 4 mm to 8 mm - two "whistle cut" openings



Sized 9 mm, 10 mm, and 12 mm - one large "scoop cut" opening



Approximately 24 cm (9.5 in) in length



Latex-free plastic

COMPATIBILITY WITH ASPIRATORS

- Compatible with Ipas MVA Plus® and Ipas Double-Valve Aspirator all sizes without adapter
- Not compatible with Ipas Single-Valve Aspirator

BENEFITS

Comfortable

 Semi-rigid and provide greater tactile feedback

Practical

• Feel the 'gritty' sensation that indicates an evacuated uterus

Easy to identify

 Cannulae are color-coded and use adapters with the same color-coding

COLOR CODE

PRODUCT CODE/DIAMETER CHART									
	Size	4 mm	5 mm	6 mm	7 mm	8 mm	9 mm	10 mm	12 mm
	Code	SR4	SR5	SR6	SR7	SR8	SR9	SR10	SR12



WomanCare Global Email: contact@dktwomancare.org www.dktwomancare.org