

Ipas Miscarriage Management


A specialized kit designed to facilitate safe and effective Manual Vacuum Aspiration (MVA) procedure




Product Code: MMK-SU

PRODUCT DESCRIPTION


KIT COMPOSITION:

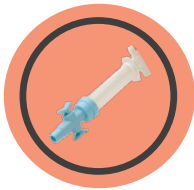

1 Ipas Double Valve Aspirator, the classic Ipas design

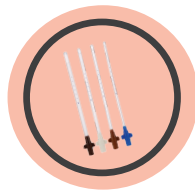

4 Ipas EasyGrip® Cannulae (6, 7, 8, and 9 mm diameters)

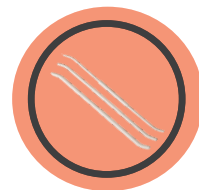

3 Denniston Dilators (5/6, 7/8, 9/10 mm diameters)

INSTRUCTION FOR USE

- Providers select the cannula size based on the woman's gestational age.
- Prepare patient and provide pain relief – oral analgesia, paracervical block.
- Creating the vacuum on the aspirator by pressing valve buttons and pulling plunger.
- Insert cannula into the uterus and attach prepared aspirator.
- Release valve buttons on the aspirator and gently rotate to aspirate uterine contents.

Scan Now: A Comprehensive Guide to the MVA Procedure



INDICATIONS FOR USE

- Miscarriage management
- Induced or incomplete abortion up to 12 weeks since the woman's last menstrual period

BENEFITS

Practical

- Ideal for emergency departments, ambulatory surgery centers and office-based settings

Ready to use

- Fully sterilized, pre-packaged, single-use kit

Convenient

- Portable, quiet and easily stored

Simple Use

- No electricity required and easy to disassemble and reassemble