## Silverline Cu 380 Ag Silverline Cu 380 Ag is a non-hormonal intrauterine device effective for up to 5 years. **CHARACTERISTICS** Works as an emergency 60 month Soft, flexible Y-shaped Silver core maintains contraceptive when frame. integrity and effectiveness (five year) shelf life. inserted up to 5 days after of copper wire over time. unprotected sex. EFFICACY MECHANISM OF ACTION Over 99% effective at • Copper is toxic to the egg and sperm, which inhibits fertilization preventing pregnancy<sup>1</sup> Silverline does not cause abortion or protect against sexually transmitted

## **ADMINISTRATION**

• Silverline Cu 380 Ag is administered by a qualified healthcare provider. Insertion is short outpatient procedure

infections.

- IUD is loaded aseptically loading to minimize infection risk using the non-contact technique
- Insertion can be performed at any point during the menstrual cycle\*
- \* if it reasonably certain the woman is not pregnant.

## BENEFITS

- Hormone Free : No side effects from estrogen or progestins
- Reversible contraceptive: Can be removed at any time with no delay in the woman's return to fertility
- Convenient: No daily action is required by the user Highly effective
- Can be inserted up to 48 hours after vaginal delivery or caesarean section\* Safe for breastfeeding women

\*if medically eligible

## CONTRAINDICATIONS

Known or suspected pregnancy, current pelvic inflammatory disease or current gynecologic infections. Women with an allergy to copper should not use Silverline Cu 380 Ag.

1. World Health Organization Department of Sexual and Reproductive Health and Research (WHO/SRH) and Johns Hopkins Bloomberg School of Public Health/ Center for Communication Programs (CCP), Knowledge SUCCESS. Family Planning: A Global Handbook for Providers (2022 update). Baltimore and Geneva: CCP and WHO; 2022.

WomanCare Global Email: contact@dktwomancare.org www.dktwomancare.org



SIDE EFFECTS

Cramping and longer, heavier

These changes are not harmful.

menstrual periods are two common

side effects of Silverline Cu 380 Ag

especially in the first 3 to 6 months.

©WomanCare Global N-TEC-SIL-701-EN, Rev.00