

DKT WOMANCARE GLOBAL PRODUCT CATALOGUE

**Creating a world where women are empowered
to live their lives to the fullest.**

| Index

SECTION I: UTERINE EVACUATION METHODS

- Ipas MVA Suite of Technology 3
- Uterine evacuation pills 15

SECTION II: CONTRACEPTIVE METHODS

- Implants 19
- Intrauterine devices 22
- Injectables 28
- Emergency pills 30



Ipas MVA Plus® Aspirator

For uterine evacuation

REUSABLE · DISCREET · SAFE · COST-EFFECTIVE



CLINICAL INDICATIONS

- Induced or incomplete abortion up to 12 weeks since the woman's last menstrual period
- Miscarriage management
- Endometrial biopsy

PRODUCT CHARACTERISTICS

- Reusable up to 25 times (where regulations allow)
- Provides holding capacity of 60 cc
- Provides a minimum vacuum of 22 inches
- Maintains vacuum for at least 30 minutes
- Cylinder marks in 10 cc increments to measure evacuated uterine contents
- Compatible with Ipas EasyGrip® Cannulae and the 12 mm Flexible Karman Cannula
- Sterilized by steam autoclavable at 250°F (121°C); or by standard cold methods (Cidex®, Sporox®II)
- High level disinfected in boiling water; or by standard cold methods (Cidex®, 0.5% Chlorine, Sporox®II)
- Latex-free plastic
- Shelf life: 5 years from month of manufacture

ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS DOUBLE VALVE ASPIRATOR

For uterine evacuation

Product Code: DVS-10 or DVS-SU



DENNISTON DILATORS

For cervical dilation associated with uterine evacuation

Product Code: DD5-S



IPAS SINGLE VALVE ASPIRATOR

For uterine evacuation

Product Code: SVS-10 or SVS-LF

Double-Valve Aspirator

For uterine evacuation and endometrial biopsy

EFFECTIVE · DISPOSABLE · AFFORDABLE · VERSATILE



CLINICAL INDICATIONS

- Induced or incomplete abortion up to 12 weeks since the woman's last menstrual period
- Miscarriage management
- Endometrial biopsy

PRODUCT CHARACTERISTICS

- 10 units sterilely packed
- Individually wrapped
- Ready to use
- Provides a holding capacity of 60 cc
- Provides a minimum vacuum of 24 inches
- Maintains vacuum for a minimum of 30 minutes
- Cylinder marks in 10 cc increments to measure evacuated uterine contents
- Compatible with Ipas EasyGrip® Cannulae and the 12 mm Flexible Karman Cannula
- Latex-free plastic
- Shelf life: 3 years from month of manufacture


ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS SINGLE VALVE ASPIRATOR

For uterine evacuation

Product Code: SVS-10 or SVS-LF

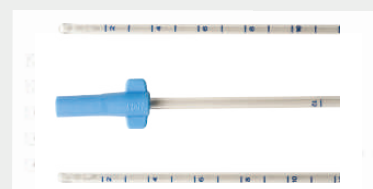

3MM CANNULA

For endometrial biopsy

Product Code: EB3



FLEXIBLE KARMAN CANNULAE

For uterine evacuation

Product Code: FC4 to FC10, and FC12

Double-Valve Aspirator

For uterine evacuation and endometrial biopsy

EFFECTIVE · DISPOSABLE · AFFORDABLE · VERSATILE



CLINICAL INDICATIONS

- Induced or incomplete abortion up to 12 weeks since the woman's last menstrual period
- Miscarriage management
- Endometrial biopsy

PRODUCT CHARACTERISTICS

- Individually packed
- Provides a holding capacity of 60 cc
- Provides a minimum vacuum of 24 inches
- Maintains vacuum for a minimum of 30 minutes
- Cylinder marks in 10 cc increments to measure evacuated uterine contents
- Compatible with Ipas EasyGrip® Cannulae and the 12 mm Flexible Karman Cannula
- Latex-free plastic
- Shelf life: 5 years from month of manufacture

ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS MVA PLUS® ASPIRATOR

For uterine evacuation
Product Code: PLUS



IPAS EASYGRIP® CANNULAE

For uterine evacuation
Product Code: SR4 to SR10, and SR12 or SR4-SU to SR10-SU, and SR12-SU



FLEXIBLE KARMAN CANNULAE

For uterine evacuation
Product Code: FC4 to FC10, and FC12

ipas Single Valve Aspirator

For uterine evacuation and endometrial biopsy

EFFECTIVE · REUSABLE · AFFORDABLE · VERSATILE



CLINICAL INDICATIONS

- Induced or incomplete abortion up to 6 weeks since the woman's last menstrual period
- Miscarriage management
- Endometrial biopsy

PRODUCT CHARACTERISTICS

- 10 units sterilely packed
- Reusable up to 25 times (where regulations allow)
- Provides a holding capacity of 60 cc
- Provides a minimum vacuum of 24 inches
- Maintains vacuum for a minimum of 30 minutes
- Cylinder marks in 10 cc increments to measure evacuated uterine contents
- Compatible with Flexible Karman Cannula sizes 4-6 mm and the Endometrial Biopsy 3MM cannula
- Latex-free plastic
- Shelf life: 3 years from month of manufacture


ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS DOUBLE VALVE ASPIRATOR

For uterine evacuation

Product Code: DVS-10 or DVS-SU

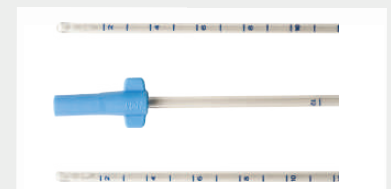

3MM CANNULA

For endometrial biopsy

Product Code: EB3



DENNISTON DILATORS

For cervical dilation associated with uterine evacuation

Product Code: DD5-S

ipas Single Valve Aspirator

For uterine evacuation and endometrial biopsy

EFFECTIVE · REUSABLE · AFFORDABLE · VERSATILE



CLINICAL INDICATIONS

- Induced or incomplete abortion up to 6 weeks since the woman's last menstrual period
- Miscarriage management
- Endometrial biopsy

PRODUCT CHARACTERISTICS

- Individually packed
- Reusable up to 25 times (where regulations allow)
- Provides a holding capacity of 60 cc
- Provides a minimum vacuum of 24 inches
- Maintains vacuum for a minimum of 30 minutes
- Cylinder marks in 10 cc increments to measure evacuated uterine contents
- Compatible with Flexible Karman Cannula sizes 4-6 mm and the Endometrial Biopsy 3MM cannula
- Latex-free plastic
- Shelf life: 3 years from month of manufacture

ALSO AVAILABLE FROM WOMANCARE GLOBAL



UTERINE PELVIC MODEL

For uterine evacuation training
Product Code: UPM



IPAS EASYGRIP® CANNULAE

For uterine evacuation
Product Code: SR4 to SR10, and SR12 or SR4-SU to SR10-SU, and SR12-SU



FLEXIBLE KARMAN CANNULAE

For uterine evacuation
Product Code: FC4 to FC10, and FC12

IPAS EasyGrip® Cannulae

The tactile response of a rigid curette with the gentle probe of a flexible cannulae

SEMI-RIGID · INNOVATIVE · VERSATILE · REUSABLE



PRODUCT CHARACTERISTICS

- Ready to use and sterile packed
- Reusable up to 25 times
- Permanently-integrated winged base
- Color-coded by size
- Uterine depth markers from 6 to 11 cm printed on cannulae
- Sized 4 mm to 8 mm - two "whistle cut" openings
- Sized 9 mm, 10 mm, and 12 mm - one large "scoop cut" opening
- Approximately 24 cm (9.5 in) in length
- Latex-free plastic
- Shelf life: 5 years from month of manufacture

COLOR CODE



PRODUCT CODE/DIAMETER CHART

Size	4 mm	5 mm	6 mm	7 mm	8 mm	9 mm	10 mm	12 mm
Code	SR4	SR5	SR6	SR7	SR8	SR9	SR10	SR12

ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS DOUBLE VALVE ASPIRATOR

For uterine evacuation
Product Code: DVS-10 or DVS-SU



IPAS MVA PLUS® ASPIRATOR

For uterine evacuation
Product Code: PLUS



UTERINE PELVIC MODEL

For uterine evacuation training
Product Code: UPM

Ipas EasyGrip® Cannulae

The tactile response of a rigid curette with the gentle probe of a flexible cannulae

SEMI RIGID · INNOVATIVE · VERSATILE · DISPOSABLE



PRODUCT CHARACTERISTICS

- Ready to use and sterile packed
- Permanently-integrated winged base
- Color-coded by size
- Uterine depth markers from 6 to 11 cm printed on cannulae
- Sized 4 mm to 8 mm - two "whistle cut" openings
- Sized 9 mm, 10 mm, and 12 mm - one large "scoop cut" opening
- Approximately 24 cm (9.5 in) in length
- Latex-free plastic
- Shelf life: 5 years from month of manufacture

COLOR CODE



PRODUCT CODE/DIAMETER CHART

Size	4 mm	5 mm	6 mm	7 mm	8 mm	9 mm	10 mm	12 mm
Code	SR4-SU	SR5-SU	SR6-SU	SR7-SU	SR8-SU	SR9-SU	SR10-SU	SR12-SU


ALSO AVAILABLE FROM WOMANCARE GLOBAL



FLEXIBLE KARMAN CANNULAE

For uterine evacuation

Product Code: FC4 to FC10, and FC12

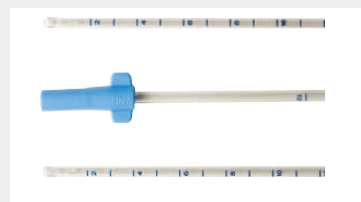

3MM CANNULA

For endometrial biopsy

Product Code: EB3



DENNISTON DILATORS

For cervical dilation associated with uterine evacuation

Product Code: DD5-S

Flexible Karman Cannulae

The original flexible cannulae

EFFECTIVE · DISPOSABLE · AFFORDABLE · FLEXIBLE



PRODUCT CHARACTERISTICS

- Sterilely packed
- Printing on cannulae color-coded by size
- Uterine depth markers from 6 to 11 cm printed on cannulae
- Sized 4 mm to 8 mm - two “whistle cut” openings
- Sized 9 mm, 10 mm, and 12 mm - one large “scoop cut” opening
- Approximately 24 cm (9.5 in) in length
- Sizes 4 mm - 6 mm compatible with Ipas Single Valve Aspirator
- Adapters required for use with Ipas MVA Plus and Double Valve Aspirator
- Latex-free plastic
- Shelf life: 5 years from month of manufacture

COLOR CODE



PRODUCT CODE/DIAMETER CHART

Size	4 mm	5 mm	6 mm	7 mm	8 mm	9 mm	10 mm	12 mm
Code	FC4	FC5	FC6	FC7	FC8	FC9	FC10	FC12

WomanCare Global Trading CIC is a UK Registered Community Interest Company (No.07231920).
Email: contact@dktwomancare.org
www.dktwomancare.org

©WomanCare Global
N-PRO-FC#*-701-EN Rev. 01

ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS MVA PLUS® ASPIRATOR
For uterine evacuation
Product Code: PLUS



DENNISTON DILATORS
For cervical dilation associated with uterine evacuation
Product Code: DD5-S




IPAS SINGLE VALVE ASPIRATOR
For uterine evacuation
Product Code: SVS-10 or SVS-LF

3MM Cannula

For endometrial biopsy

EFFECTIVE · DISPOSABLE · AFFORDABLE · VERSATILE

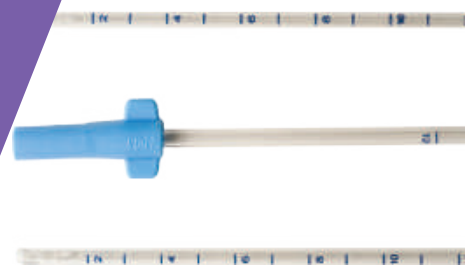

CLINICAL APPLICATIONS

- Evaluation of infertility
- Evaluation of amenorrhea
- Evaluation of suspected endometrial cancer
- Evaluation of abnormal uterine bleeding
- Diagnosis of endometrial infections

PRODUCT CHARACTERISTICS

- Disposable and sterile packed
- Permanently integrated winged base
- Two opposing apertures
- Approximately 23 cm (9 in) in length
- Depth markings printed in cm
- Compatible with Ipas MVA Plus® Aspirator with a 6 mm adaptor
- Compatible with Ipas MVA Single Valve Aspirator
- Medical grade, latex-free copolyester plastic
- Shelf life: 5 years from month of manufacture

COLOR CODE

3mm

WomanCare Global Trading CIC is a UK Registered Community Interest Company (No.07231920).
Email: contact@dktwomancare.org
www.dktwomancare.org

©WomanCare Global
N-PRO-EB3-701-EN Rev. 01

ALSO AVAILABLE FROM WOMANCARE GLOBAL



UTERINE PELVIC MODEL

For uterine evacuation training
Product Code: UPM



FLEXIBLE KARMAN CANNULAE

For uterine evacuation
Product Code: FC4 to FC10, and FC12



IPAS MVA PLUS® ASPIRATOR

For uterine evacuation
Product Code: PLUS

Denniston Dilators

For cervical dilation

REUSABLE · GRADUATED · LIGHTWEIGHT



CLINICAL INDICATION

- For cervical dilation prior to uterine evacuation

PRODUCT CHARACTERISTICS

- Packed sterile
- Tapered - requires half the force of a non-tapered dilator
- Set of 5 double-ended, tapered dilators: 5/6 mm, 7/8 mm, 9/10 mm, 11/12 mm, and 13/14 mm diameters
- Approximately 29 cm (11.5 in) in length
- Latex-free, polypropylene plastic
- Sterilized by steam autoclavable at 250°F (121°C) or by standard cold methods (Cidex®, Sporox® II)
- High level disinfected in boiling water; or by standard cold methods (Cidex®, 0.5% Chlorine, Sporox® II)
- Shelf life: 5 years from month of manufacture

ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS MVA PLUS® ASPIRATOR
For uterine evacuation
Product Code: PLUS



IPAS EASYGRIP® CANNULAE
For uterine evacuation
Product Code: SR4 to SR10, and SR12
or SR4-SU to SR10-SU, and SR12-SU



IPAS SINGLE VALVE ASPIRATOR
For uterine evacuation
Product Code: SVS-10 or SVS-LF

Uterine Pelvic Model

For effective training

SIMPLE TO USE · OPEN DESIGN · REALISTIC



PRODUCT CHARACTERISTICS

- Open design for ease of intrauterine procedure training, including uterine evacuation
- Simulation of both smooth and gritty tactile sensations during uterine evacuation
- Plexiglass base with suction cups to ensure stability
- Lightweight and portable
- Measures 25 x 25 x 18 cm (10 x 10 x 10 in) approximately



ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS DOUBLE VALVE ASPIRATOR

For uterine evacuation

Product Code: DVS-10 or DVS-SU



DENNISTON DILATORS

For cervical dilation associated with uterine evacuation

Product Code: DD5-S



IPAS SINGLE VALVE ASPIRATOR


For uterine evacuation

Product Code: SVS-10 or SVS-LF

Miscarriage Management

For early pregnancy loss management

COST-EFFECTIVE · CONVENIENT · EFFICIENT

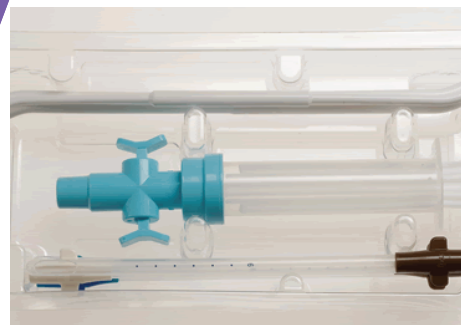


PRODUCT CHARACTERISTICS

- Fully sterilized, pre-packaged, single-use kit
- Hand-held, simple, and portable
- Affordable alternative to surgical operating room procedures
- Ideal for emergency departments, ambulatory surgery centers and office-based setting

KIT COMPOSITION

- 1 Ipas Double Valve Aspirator, the classic Ipas design
- 4 Ipas EasyGrip® Cannulae (6, 7, 8, and 9 mm diameters)
- 3 Denniston Dilators (5/6, 7/8, 9/10 mm diameters)

ALSO AVAILABLE FROM WOMANCARE GLOBAL

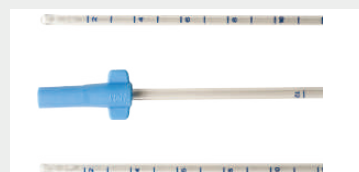

3MM CANNULA

For endometrial biopsy
Product Code: EB3



IPAS EASYGRIP® CANNULAE

For uterine evacuation
Product Code: SR4 to SR10, and SR12
or SR4-SU to SR10-SU, and SR12-SU



DENNISTON DILATORS

For cervical dilation associated
with uterine evacuation
Product Code: DD5-S

MISO-FEM®

A versatile product with multiple applications

HIGH-QUALITY · COST-EFFECTIVE · EASY-TO-USE



HOW IT WORKS

- **Stimulates the smooth muscle** to cause uterine contractions
- **Softens the cervix** through disintegration and dissolution of collagen
- **Dilates blood vessels** to reduce blood pressure in the uterus
- **Can be administered orally, buccally, or vaginally**

OBSTETRIC AND GYNECOLOGICAL USES

- **Medical abortion:** effective both alone (87%) and in combination with mifepristone (96%)¹ where allowed
- **Miscarriage management:** effective and highly acceptable for treating early pregnancy loss or incomplete abortion
- **Cervical ripening:** effective ripening agent prior to first-trimester aspiration abortions. Also used prior to endometrial biopsy or IUD insertion
- **Labor induction:** highly effective for inducing vaginal delivery within 24 hours
- **Postpartum hemorrhage (PPH):** used for prevention and treatment of PPH

Misoprostol compares favorably to oxytocin for PPH because it is inexpensive, thermostable, has low side-effects rates, and does not require injection²

1. Kapp, N., & Lohr, P. A. (2020). Modern methods to induce abortion: Safety, efficacy and choice. Best Practice & Research Clinical Obstetrics & Gynaecology, 63, 37-44. doi: 10.1016/j.bpobgyn.2019.11.008

2. Allen, R., & O'Brien, B. M. (2009). Uses of misoprostol in obstetrics and gynecology. Reviews in obstetrics & gynecology, 2(3), 159-168.

ALSO AVAILABLE FROM
WOMANCARE GLOBAL



MISOMIFE-FEM® COMBO

For uterine evacuation
Combi pack of 1 mifepristone 200 mg tablet
and 4 misoprostol 200 mcg tablets
Product Code: COMBO



POSTPIL®

Pack of 1 levonorgestrel 1.5 mg tablet
Product Code: EC1



IPAS MVA PLUS® ASPIRATOR

For uterine evacuation
Product Code: PLUS

MISO-FEM®

A versatile product with multiple applications

HIGH QUALITY · COST-EFFECTIVE · EASY-TO-USE




INSTRUCTIONS FOR UTERINE EVACUATION: STEP 1

Administer 4 misoprostol pills:

- **Buccally**, between your cheek and bottom gums for 30 minutes, then swallow with water
- **Sublingually**, under your tongue for 30 minutes, then swallow with water

Suggestions for use:

- Have someone with you for support
- Before taking the pills, take a drink of water to moisten your mouth
- Do not eat or drink anything once you take the pills
- You may experience cramps and bleeding
- Expect to pass clots of blood and tissue

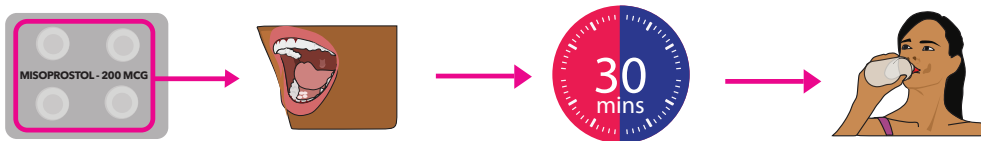

INSTRUCTIONS FOR UTERINE EVACUATION: STEPS 2 & 3

- Wait 3-4 hours
- Repeat step 1 with another 4 pills
- Wait 3-4 hours
- Repeat step 1 with another 4 pills

ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS DOUBLE VALVE ASPIRATOR

For uterine evacuation
Product Code: DVS-10 or DVS-SU



IPAS EASYGRIP® CANNULAE

For uterine evacuation
Product Code: SR4 to SR10, and SR12 or SR4-SU to SR10-SU, and SR12-SU



FLEXIBLE KARMAN CANNULAE

For uterine evacuation
Product Code: FC4 to FC10, and FC12

PRODUCT CODE: COMBO

MISOMIFE-FEM® COMBO

Pack of 1 mifepristone 200 mg tablet and 4 misoprostol 200 mcg tablets

AN EFFECTIVE, HIGH-QUALITY, SAFE, AND DISCREET
OPTION FOR UTERINE EVACUATION

WHAT IS MISOMIFE-FEM® COMBO

- **Mifepristone** 200 mg (1 tablet) stops the action of the hormone that sustains pregnancy (progesterone). This causes the embryo to detach from the uterus and the pregnancy to stop growing
- **Misoprostol** 800 mcg (4 tablets) causes contraction, to push out the contents of the uterus

OBSTETRIC AND GYNECOLOGICAL USES

Ensures complete uterine evacuation:

- For first-trimester abortion or miscarriage
- When therapeutic abortion is required for a pregnancy that is 9 weeks or less
- For treatment of postpartum hemorrhage / Post Abortion Care (PAC)
- To prevent a hysterectomy and shorten the duration of conservative management of placenta accreta



- MORE EFFECTIVE THAN USING MISOPROSTOL ALONE¹
- SAFE METHOD FOR UTERINE EVACUATION
- CAN BE USED OUTSIDE THE HOSPITAL, EVEN AT HOME (WHERE ALLOWED)
- DISCREET AND RESPECTFUL OF THE WOMAN'S PRIVACY
- AVAILABLE FROM PHARMACY (WHERE ALLOWED)
- PREVENTS MATERNAL DEATHS BY AVERTING UNSAFE ABORTION
- REGIMEN RECOMMENDED BY THE WHO

ALSO AVAILABLE FROM
WOMANCARE GLOBAL



MISO-FEM®

For uterine evacuation

Pack of 4 misoprostol 200 mcg tablets

Product Code: MISO



POSTPIL®

Pack of 1 levonorgestrel 1.5 mg tablet

Product Code: EC1



IPAS MVA PLUS® ASPIRATOR

For uterine evacuation

Product Code: PLUS

1. Kapp, N., & Lohr, P. A. (2020). Modern methods to induce abortion: Safety, efficacy and choice. Best Practice & Research Clinical Obstetrics & Gynaecology, 63, 37-44. doi: 10.1016/j.bpobgyn.2019.11.008

MISOMIFE-FEM® COMBO

Pack of 1 mifepristone 200 mg tablet and 4 misoprostol 200 mcg tablets

AN EFFECTIVE, HIGH-QUALITY, SAFE, AND DISCREET
OPTION FOR UTERINE EVACUATION




INSTRUCTIONS FOR UTERINE EVACUATION: STEP 1

Swallow mifepristone tablet with water

- Patient should have someone with them for support
- They should eat something light (e.g. crackers) or take a drug to prevent vomiting (ask for an "anti-emetic" drug)

Wait for 36 hours after taking misoprostol tablet

- Most women do not feel any symptom during this period
- Some may feel like vomiting or have slight bleeding

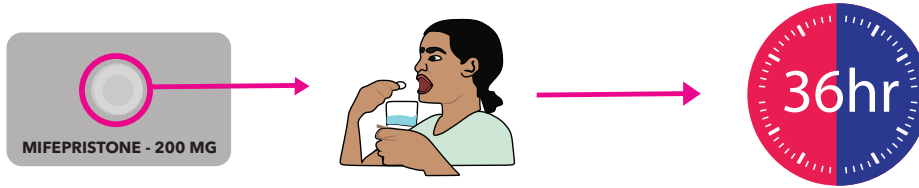


INSTRUCTIONS FOR UTERINE EVACUATION: STEP 2

Administer 4 tablets of 200 mcg misoprostol (800 mcg total)

- **Buccally**, between your cheek and bottom gums for 30 minutes then swallow with water
- **Sublingually**, under your tongue for 30 minutes then swallow with water

What to expect:

- Clients should expect to have painful cramps and bleeding
- They should expect to pass clots of blood and tissue

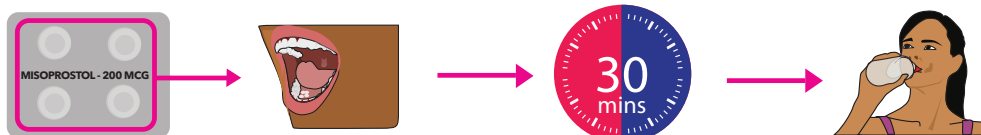

ALSO AVAILABLE FROM WOMANCARE GLOBAL



INJECTA-FEM

Intramuscular contraceptive injection
Medroxyprogesterone acetate 150 mg/ml
Product Code: INJECTA



LEVOPLANT™

WHO-prequalified
2-rod hormonal implant
Product Code: LEVO



UTERINE PELVIC MODEL

For uterine evacuation training
Product Code: UPM

LEVOPLANT™

SUBDERMAL CONTRACEPTIVE IMPLANT

A REVERSIBLE, LONG-ACTING, EFFECTIVE,
AND SAFE CONTRACEPTIVE OPTION

EFFECTIVENESS

- One of the most effective contraceptive methods available
- Effectiveness comparable to that of surgical sterilization
- Cumulative pregnancy rate in clinical trials was 0.2% at 3 years*
- Set it and forget it method that prevents misuse

BENEFITS OF LONG-ACTING, REVERSIBLE CONTRACEPTIVE METHODS

- **Reliable yet easy to use:** no daily action needed
- **Helps couples achieve their desired family size**
- **Offers complete control:** can begin use at any time in the menstrual cycle and remove whenever a pregnancy is desired

- HIGHLY EFFECTIVE
- LONG-ACTING
- REVERSIBLE
- RELIABLE
- EASY TO USE
- DISCREET
- SIMPLE INSERTION AND REMOVAL
- NO ESTROGEN – AVOIDS ESTROGEN SIDE EFFECTS
- LOW COST
- IDEAL FOR SPACING OR LIMITING BIRTHS
- NO DELAY IN A WOMAN'S RETURN TO FERTILITY AFTER REMOVAL

PRODUCT PORTFOLIO

ASPIRATORS

- Ipas Plus® Aspirator
- Single Valve Aspirator
- Double Valve Aspirator

CANNULAE

- Ipas EasyGrip® Cannulae
- Flexible Karman Cannulae

DILATORS AND ACCESSORIES

- Adapter sets, dilators, and aspirator accessories also available

KITS

- Miscarriage Management Kit

TRAINING TOOLS

- Pelvic Model

* World Health Organization. "WHOPAR, Part 4: Summary of Product Characteristics." Prequalified medicines/finished pharmaceutical product lists, reference number RH028. Published 30 June 2017. <https://extranet.who.int/prequal/medicine/3749>

LEVOPLANT™

SUBDERMAL CONTRACEPTIVE IMPLANT

A REVERSIBLE, LONG-ACTING, EFFECTIVE,
AND SAFE CONTRACEPTIVE OPTION

DISPOSABLE
TROCAR



2 SILICONE
ELASTOMER
RODS

PRODUCT CHARACTERISTICS

- **High-quality, affordable 2-rod contraceptive implant**
- **75 mg of the progestin levonorgestrel** in each rod (150 mg total)
- **Very low dose of levonorgestrel** - lower metabolized progestin levels than oral or injectable contraceptives
- **Contains no estrogen** and avoids estrogen-related side effects
- **Inserted just under the skin of a woman's upper arm** by a trained healthcare provider using a disposable trocar
- **60-month (five-year) shelf life** when stored at $\leq 30^{\circ}\text{C}$



ALSO AVAILABLE FROM WOMANCARE GLOBAL



MISOMIFE-FEM® COMBO

For uterine evacuation
Combi pack of 1 mifepristone 200 mg tablet
and 4 misoprostol 200 mcg tablets
Product Code: COMBO



MISO-FEM®

For uterine evacuation
Pack of 4 misoprostol 200 mcg tablets
Product Code: MISO



POSTPIL®

Pack of 1 levonorgestrel 1.5 mg tablet
Product Code: EC1

UPPER ARM MODEL

For effective training

EASY TO USE · REALISTIC · REUSABLE



PRODUCT CHARACTERISTICS

- **High-quality device** developed to train clinicians on how to insert the implant
- **Includes muscle, subcutaneous fat, and skin layers** on both ends to facilitate insertion at proper depth
- **Detachable top skin layers and subcutaneous fat** allows for multiple insertions and checking the accuracy of insertion depth and also be shown to the patient for educational purposes



ALSO AVAILABLE FROM WOMANCARE GLOBAL



IPAS MVA PLUS® ASPIRATOR
For uterine evacuation
Product Code: PLUS



IPAS EASYGRIP® CANNULAE
For uterine evacuation
Product Code: SR4 to SR10, and SR12
or SR4-SU to SR10-SU, and SR12-SU



IPAS SINGLE VALVE ASPIRATOR
For uterine evacuation
Product Code: SVS-10 or SVS-LF

PRODUCT CODE: ELOIRA

Eloira

HORMONAL INTRAUTERINE SYSTEM (IUS)

HIGHLY-EFFECTIVE · REVERSIBLE · LONG-TERM · AFFORDABLE



WHO CAN USE

Can be used by most sexually active women:

- Single or married women who want a highly effective long-term option for preventing pregnancy
- Have or have not had children
- Breastfeeding mothers

WHO CANNOT USE

Women with:

- Current or history of breast cancer
- Liver disease or tumor
- Kidney disease or tumor
- Severe uncontrolled hypertension
- Untreated pelvic infection or cancer of the cervix or uterus

HOW IT WORKS

- Rate-controlling membrane releases 20 µg of levonorgestrel (LNG) in the uterine cavity per day
- Thickens the mucus inside the cervix
- Inhibits sperm function
- Reduces growth of endometrial lining

OTHER LONG ACTING CONTRACEPTIVES

ETHERENA

INTRAUTERINE ENABLERS (IUE)

- Single-hand procedure
- Ergonomically designed
- 2-step operation



SILVERLINE

INTRAUTERINE DEVICES (IUD)

- More options for women
- No hormone-related side effects
- Low discomfort



LEVOPLANT™

2-ROD HORMONAL IMPLANT

- WHO-prequalified
- Reliable yet easy to use



Eloira

HORMONAL INTRAUTERINE SYSTEM (IUS)

HIGHLY-EFFECTIVE · REVERSIBLE · LONG-TERM · AFFORDABLE



BENEFITS FOR THE PROVIDER

- **Versatile, cost-effective solution** treats variety of conditions
- **High client satisfaction** reflected in high continuation rates
- **More affordable** than other hormonal IUDs
- **Flexible Y-frame design** easy to load and insert

BENEFITS FOR THE PATIENT

- **Highly effective contraception** with minimal side effects due to localized hormonal concentration
- Also effective in treating effects of:
 - **Menorrhagia**: heavy or prolonged menstrual bleeding
 - **Dysmenorrhea**: painful menstrual cramping
 - **Endometriosis**: abnormal growth of endometrial tissue - causes pelvic pain
 - **Adenomyosis**: abnormal growth of uterine lining - causes pain and bleeding
 - **Anemia**: decrease in red blood cells in the bloodstream
 - **Uterine fibroids**: cellular growths in or on the uterus
- Effective **substitute for hysterectomy**:
 - 64% of women awaiting hysterectomy operation canceled after 6 months of using the hormonal IUS
 - Provides safe, quick, affordable outpatient alternative to surgery
- Does not interfere with sexual activity

OTHER LONG ACTING CONTRACEPTIVES

ETHERENA

INTRAUTERINE ENABLERS (IUE)


- Single-hand procedure
- Ergonomically designed
- 2-step operation



SILVERLINE

INTRAUTERINE DEVICES (IUD)

- More options for women
- No hormone-related side effects
- Low discomfort

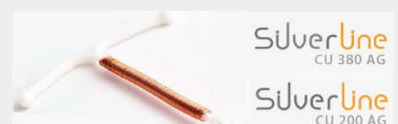

LEVOPLANT™

2-ROD HORMONAL IMPLANT

- WHO-prequalified
- Reliable yet easy to use





Etherena

INTRAUTERINE ENABLERS (IUE)

ACCURATE · INTUITIVE · SAFE · ERGONOMIC HANDLING

WHO CAN USE

Can be used by most sexually active women:

- Single or married women who want an effective long-term option for preventing pregnancy
- Breastfeeding mothers
- Women with certain health conditions who are not allowed to use hormonal contraceptive
- Women who do not want to use hormonal contraceptives

WHO CANNOT USE

Women with:

- Current or history of breast cancer
- Liver disease or tumor
- Kidney disease or tumor
- Severe uncontrolled hypertension
- Untreated vaginal infection

HOW IT WORKS

- Physical barrier prevents the sperm from fertilizing the egg
- Copper prevents implantation so pregnancy does not occur

OTHER LONG ACTING CONTRACEPTIVES

ELOIRA

INTRAUTERINE SYSTEM (IUS)


- Hormonal method
- Treats numerous conditions
- Substitute for hysterectomy



SILVERLINE

INTRAUTERINE DEVICES (IUD)

- More options for women
- No hormone-related side effects
- Low discomfort

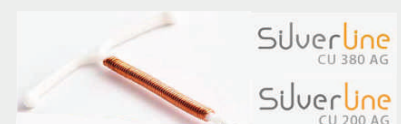

LEVOPLANT™

2-ROD HORMONAL IMPLANT

- WHO-prequalified
- Reliable yet easy to use



Etherena

INTRAUTERINE ENABLERS (IUE)

ACCURATE · INTUITIVE · SAFE · ERGONOMIC HANDLING



BENEFITS FOR THE PROVIDER

Innovative Intra-Uterine Enablers (IUE) technology offers:

- No-touch aseptic procedure
- Easy one-handed loading and insertion
- Perfect fundal placement reduces pain, bleeding, and risk of expulsion
- Curvilinear design fits retroverted or anteverted uterus
- Quicker, easier insertion reduces fatigue and shortens procedure time
- Intuitive click sound adjusts per depth of uterus
- In-built uterine scale for operability, convenience, and accuracy
- Comes with complementary uterine sound in sterile package

BENEFITS FOR THE PATIENT

- **Highly effective contraception** (99.9%) for up to 10 years
- **Safe and quick procedure** with less waiting time
- **Natural method** with no hormonal side effects
- **Immediate return to fertility** upon removal
- **Can be inserted at any time** during menstrual cycle
- **No daily attention** and does not interfere with sexual activity
- **Soft nylon suture** is comfortable for both partners
- **Emergency contraception** when inserted within 120h (5 days) after unprotected sex

OTHER LONG ACTING CONTRACEPTIVES

ELOIRA

INTRAUTERINE SYSTEM (IUS)

- Hormonal method
- Treats numerous conditions
- Substitute for hysterectomy



SILVERLINE

INTRAUTERINE DEVICES (IUD)

- More options for women
- No hormone-related side effects
- Low discomfort



LEVOPLANT™

2-ROD HORMONAL IMPLANT

- WHO-prequalified
- Reliable yet easy to use




PRODUCT CODE: SLVR380/SLVR200

Silverline

INTRAUTERINE DEVICE (IUD)

EFFECTIVE · COMFORTABLE · CONVENIENT · SAFE

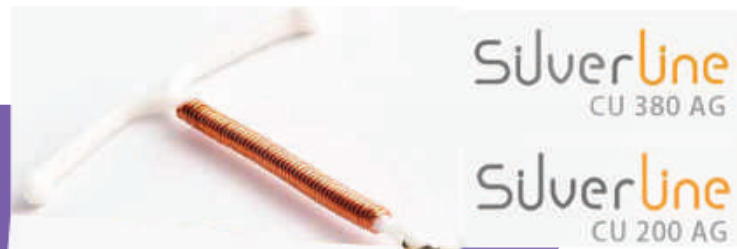

WHO CAN USE

Can be used by most sexually active women:

- Single or married women who want an effective long-term option for preventing pregnancy
- Breastfeeding mothers
- Women with certain health conditions who are not allowed to use hormonal contraceptive
- Women who do not want to use hormonal contraceptives

WHO CANNOT USE

Women with:

- Current or history of breast cancer
- Liver disease or tumor
- Kidney disease or tumor
- Severe uncontrolled hypertension
- Untreated vaginal infection

HOW IT WORKS

- Physical barrier prevents the sperm from fertilizing the egg
- Copper prevents implantation so pregnancy does not occur

OTHER LONG ACTING CONTRACEPTIVES

ELOIRA

INTRAUTERINE SYSTEM (IUS)

- Hormonal method
- Treats numerous conditions
- Substitute for hysterectomy



ETHERENA

INTRAUTERINE ENABLERS (IUE)

- Single-hand procedure
- Ergonomically designed
- 2-step operation



LEVOPLANT™

2-ROD HORMONAL IMPLANT

- WHO-prequalified
- Reliable yet easy to use




PRODUCT CODE: SLVR380/SLVR200

Silverline

INTRAUTERINE DEVICE (IUD)

EFFECTIVE · COMFORTABLE · CONVENIENT · SAFE

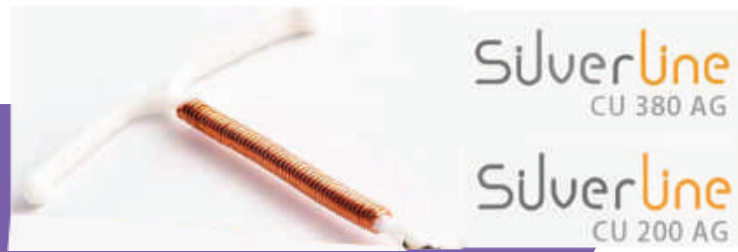

BENEFITS FOR THE PROVIDER

- **Soft, flexible Y-frame design** offers perfect fundal placement which reduces risk of perforation or expulsion
- **Silver core** prevents copper wire from fragmenting
- **Quicker insertion** as requires only 1/3 dilation of Copper T 380A
- **Printed insertion** tube adds accuracy and convenience
- **Packed sterile** to prevent infection

BENEFITS FOR THE PATIENT

- **Highly effective contraception (99.9%)**
 - 5 years of effectiveness with Silverline Cu 380 Ag
 - 3 years of effectiveness with Silverline Cu 200 Ag
- **Quicker, more comfortable insertion** requires only 1/3 dilation of standard IUD
- **Natural method** with no hormonal side effects
- **Lower copper content** better for women with anemia or excess bleeding condition
- **Immediate return to fertility** upon removal
- **Can be inserted at any time** during menstrual cycle
- **No daily attention** and does not interfere with sexual activity
- **Soft nylon suture** is comfortable for both partners
- **Emergency contraception** when inserted within 120h (5 days) after unprotected sex

OTHER LONG ACTING CONTRACEPTIVES

ELOIRA

INTRAUTERINE SYSTEM (IUS)

- Hormonal method
- Treats numerous conditions
- Substitute for hysterectomy



ETHERENA

INTRAUTERINE ENABLERS (IUE)

- Single-hand procedure
- Ergonomically designed
- 2-step operation



LEVOPLANT™

2-ROD HORMONAL IMPLANT

- WHO-prequalified
- Reliable yet easy to use



INJECTA-FEM

Medroxyprogesterone Acetate 150 mg/ml suspension for injection

EFFECTIVE · NON-INVASIVE · CONVENIENT · QUICK



WHAT IS INJECTA-FEM

- **Releases progestin hormone** into the bloodstream
- **Administered by an injection** in the gluteal or deltoid muscle every 3 months by healthcare provider with immediate effectiveness
- **Mechanisms of action:** stops ovulation, inhibits sperm transport through the fallopian tube, and prevents implantation

WHO CAN USE INJECTA-FEM

Most sexually active women:

- Women looking for a **highly effective long-acting contraceptive option to limit or space births**
- **HIV-positive** or women using **antiretroviral (ARV) therapy**
- **Breastfeeding** women

WHO CANNOT USE INJECTA-FEM

Women with current or history of:

- Breast cancer
- Abnormal vaginal bleeding
- Risks factors for osteoporosis
- Severe liver dysfunction
- Conditions that could increase chances of heart disease, such as high blood pressure, blood clots in legs or lungs, or diabetes

OTHER DKT WOMANCARE PRODUCTS

IPAS MVA PLUS® ASPIRATOR

FOR UTERINE EVACUATION

- WHO-recommended method
- Reusable 25 times (where allowed)



IPAS EASYGRIP® CANNULAE

FOR UTERINE EVACUATION

- Semi-rigid
- Reusable 25 times (where allowed)



LEVOPLANT™

2-ROD HORMONAL IMPLANT

- WHO-prequalified
- Reliable yet easy to use



INJECTA-FEM

Medroxyprogesterone Acetate 150 mg/ml suspension for injection

EFFECTIVE · NON-INVASIVE · CONVENIENT · QUICK



BENEFITS FOR THE PROVIDER

- **Quick procedure**
- **Simple injection requires only basic skills**
- **Procedure does not require specialized clinical setup or equipment**
- **Does not require** prior pelvic examination, routine blood tests, cervical cancer screening, or breast examination

BENEFITS FOR THE PATIENT

- **> 99% effective**
- **Discreet and private contraceptive method**
- **Long-acting duration means no daily or monthly routine required**
- **Reduces symptoms of endometriosis,** like pelvic pain and irregular bleeding*
- **Less invasive procedure** than for other long-acting methods
- **Avoids estrogenic side effects**
- **Helps protect against endometrial cancer and uterine fibroids****
- **Can reduce or stop monthly bleeding,** which is not harmful and may help prevent anemia**

*World Health Organization Department of Reproductive Health and Research (WHO/RHR) and Johns Hopkins Bloomberg School of Public Health/Center for Communication Programs (CCP), Knowledge for Health Project. Family Planning: A Global Handbook for Providers (2018 update). Baltimore and Geneva: CCP and WHO, 2018.

WomanCare Global Trading CIC is a UK Registered Community Interest Company (No.07231920)
Email: contact@dktwomancare.org
www.dktwomancare.org

©WomanCare Global
N-TC-INJECTA-701-EN Rev. 00

OTHER DKT WOMANCARE PRODUCTS

ETHERENA

INTRAUTERINE ENABLERS (IUE)

- Single-hand procedure
- Ergonomically designed
- 2-step operation



SILVERLINE

INTRAUTERINE DEVICES (IUD)

- More options for women
- No hormone-related side effects
- Low discomfort



ELOIRA

INTRAUTERINE SYSTEM (IUS)


- Hormonal method
- Treats numerous conditions
- Substitute for hysterectomy



POSTPIL®

Effective and convenient 1-pill solution for emergency contraceptive

EASY-TO-USE · EASY-TO-FIND · EFFECTIVE · SAFE

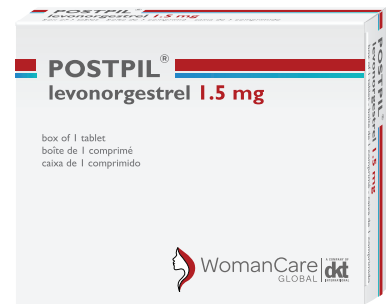

WHAT IS POSTPIL®

- **Emergency contraceptive** to use after unprotected intercourse or in case of known or suspected contraceptive failure
- **Package contains 1 tablet of levonorgestrel 1.5 mg**
- **Not an abortion pill**; does not terminate established pregnancy
- **Does not protect against sexually transmitted infections (STIs)**
- **Debunking common myths:**
 - No health risks with repeated uses
 - Does not affect fertility
 - Does not increase the risk of ectopic pregnancy, cancer or cause long term side effects

ALSO AVAILABLE FROM
WOMANCARE GLOBAL



MISOMIFE-FEM® COMBO

Combi pack of 1 mifepristone 200 mg tablet and 4 misoprostol 200 mcg tablets
Product Code: COMBO

WHO CAN USE POSTPIL®

- **Women of all reproductive ages who want to avoid pregnancy**, including adolescents, women who are breastfeeding, or women with history of ectopic pregnancy
- **No absolute contraindications**, except hypersensitivity to levonorgestrel

WHY USE POSTPIL®

- **WHO-recommended regimen** to prevent pregnancy
- **Meets European standards and certifications** for quality control of medicines and production processes
- **1-pill formulation** is easier to follow correctly than 2-pill formulation
- **High-quality yet affordable product**



INJECTA-FEM

Intramuscular contraceptive injection
Medroxyprogesterone acetate 150 mg/ml
Product Code: INJECTA




IPAS DOUBLE VALVE ASPIRATOR

For uterine evacuation
Product Code: DVS-10 or DVS-SU

POSTPIL®

Effective and convenient 1-pill solution for emergency contraceptive

EASY-TO-USE · EASY-TO-FIND · EFFECTIVE · SAFE

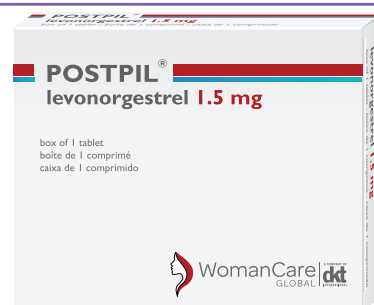

WHEN TO USE POSTPIL®

- **Can be used at any time during the menstrual cycle for the following situations:**
 - Desire to avoid pregnancy
 - Nonconsensual or forced unprotected sex
 - Possible contraceptive failure or incorrect use
 - Concern about the withdrawal method
 - Uncertainty about the 'safe period'
- **Can be used if already administered** in the same menstrual cycle

HOW POSTPIL® WORKS

- **Disrupts ovulation and thickens the cervical mucus**, which inhibits sperm mobility and prevents fertilization
- **No examinations or laboratory tests needed**¹
- **Reduces risk of pregnancy by 80%-90%**
- **Side effects may include:** nausea, lower abdominal pain, fatigue, headache, dizziness, breast tenderness, delay of menses (>7 days)

HOW TO USE POSTPIL®

- **To be used as soon as possible up to 3 days (72 hours)** after unprotected intercourse for best results. Postpil® is more effective the sooner it is taken after unprotected sex.
- **Dosage should not be taken more than once within 24 hours**, except if the woman vomits within 2 hours after use
- **Risk of pregnancy if repeated unprotected intercourse** within 24 hours after taking the Postpil®

¹ Rodrigues, Isabel & Grou, Fabienne & Joly, Jacques. (2001). Effectiveness of emergency contraceptive pills between 72 and 120 hours after unprotected sexual intercourse. American journal of obstetrics and gynecology. 184. 531-7. 10.1067/mob.2001.111102. 1

ALSO AVAILABLE FROM
WOMANCARE GLOBAL



LEVOPLANT™

WHO-prequalified 2-rod hormonal implant
Product Code: LEVO



MISO-FEM®

For uterine evacuation
Pack of 4 misoprostol 200 mcg tablets
Product Code: MISO



IPAS MVA PLUS® ASPIRATOR
Uterine evacuation device
Product Code: PLUS



WomanCare | A COMPANY OF GLOBAL **dkt** INTERNATIONAL

**CREATING A WORLD WHERE WOMEN ARE EMPOWERED
TO LIVE THEIR LIVES TO THE FULLEST**

©Copyright WomanCare Global Trading CIC | contact@dktwomancare.org | www.dktwomancare.org

WomanCare Global
1640 Roanoke Blvd,
Salem, VA 24153, USA

WomanCare Global Trading CIC
One Bartholomew Close,
EC1A 7BL, London, UK

DKT WomanCare Global Services SAS
28 Boulevard Haussmann
75009 Paris, France